

PONTE GIULIO®

ASSEMBLY INSTRUCTIONS

FOLDING SEAT

ROME SERIE G25




 PRODUCT CONFORM TO MDD93/42 EEC

 ANTIMICROBIAL

 RECYCLABLE PRODUCT

Thank you for purchasing this Ponte Giulio item. This product has been manufactured in respect of the safety requirements provided by law

Ponte Giulio USA, Inc
W241 S4021 Rockwood Circle Waukesha
WI 53189 – USA
Phone +1.877.766.8301 * fax +1.877.766.8306
Web Site: www.pontegiuliousa.com
Email: pontegiulio@sbcglobal.net

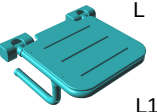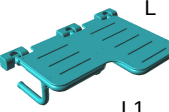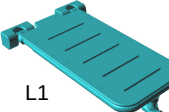
CODE		L x L1	e	
G25JDSxx20		17" x 15 1/2"	14.226	450
G25JDSxx21		17" x 18 1/2"	16.534	450
G25JDxxx28		21 1/2" x 26 7/8"	27.249	450
G25JDSxx37		13" x 27 3/16"	15.873	450
G25JDSxx38		13" x 31 1/8"	17.341	450

 = Lbs = Rated load


e = Lbs = Net Weight

 = Read the manual carefully


1

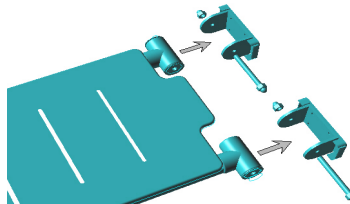


2

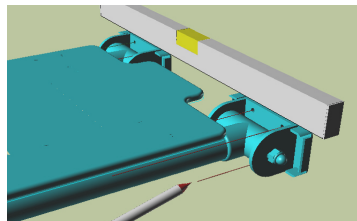


3

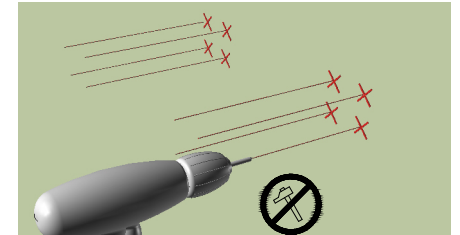


4

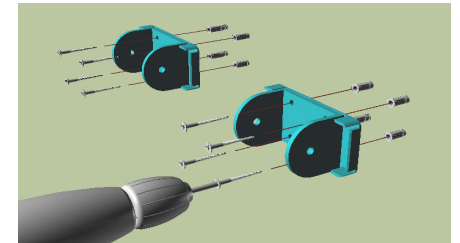


5

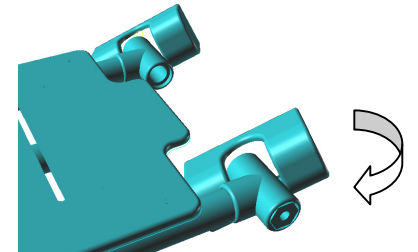

6

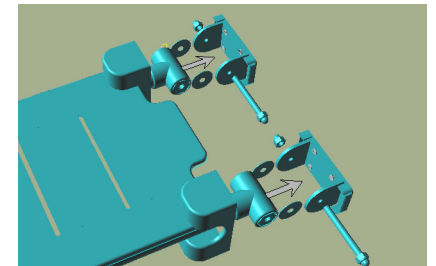

7

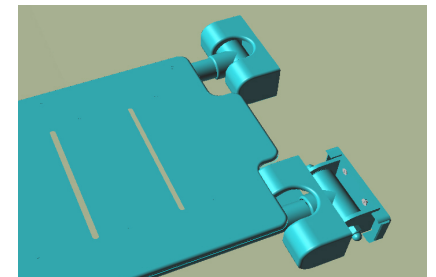

LIMITED WARRANTY

If this product fails due to a defect in material or workmanship during the time of periods listed below for each product, Ponte Giulio USA Corp. will replace or repair it free of charge. Simply contact Ponte Giulio at +1 877 766 8301 for replacement information.

GRAB BAR = 5 YEARS

EXCEPTIONS AND RESTRICTIONS

This warranty does not cover products which have not been installed or used in accordance with the instructions concerning installation and use set forth in the owner's manual or installation instruction, and when the recommendations and warning included are not observed.

This warranty does not cover products that have been modified or dismantled partially or totally, or have been used in a negligent fashion and as consequence has suffered damages attributable to the consumer, individual, or hardware not authorized by Ponte Giulio.

This warranty does not cover products which have been abused, damaged, altered, misused, cut or worn, making substitutions or modifying this product in any way will void your warranty.

This warranty does not cover the damages as a result of disaster such as fire or acts of God, including flooding, earthquake, or electric storms, etc.

LIMITATION OF LIABILITY

Other than the limited warranty stated above, to the extent permitted by law, Ponte Giulio disclaims all other implied or express warranties, including all warranties of merchantability and/or fitness for a particular purpose. This warranty is your sole remedy and Ponte Giulio shall not be liable for any damages (whether direct, incidental, special, consequential, exemplary or otherwise, including lost revenues and lost profits) arising out of any theory of recovery, including statutory, contract or tort. Notwithstanding the term of any limited or implied warranty, or in the event that any limited or implied warranty, or in the event that any limited warranty fails of its essential purpose, in no event will Ponte Giulio's entire liability exceed the purchase price of this product, some states and provinces do not allow the exclusion or limitation of incidental or consequential damages so the above limitations or exclusions may not apply to you this limited warranty gives you specific legal rights, and you may also have other rights which vary from state to state and province to province. Ponte Giulio assumes no liability for any personal injury or property damage caused by improper assembly, installation or use of this product.

INSTALLATION

The installation of the safety support must be carried out only by competent and qualified personnel who will have to use the most suitable fixing screws according to the various types of walls. Installers must make sure that the wall by the fixing plate is perfectly flat, without any protruding joints or tiles, that the wall's structure is suitable for fixing the safety support and that there are no weak points. Only use 3 screws for each fixing plate. More than 3 screws weaken the wall. The omission or non efficiency of one screw will dangerously reduce the safety support's load resistance. The omission or non efficiency of 2 or more screws does not guarantee the safety support's resistance. The screws in the packaging are suitable exclusively for the following kinds of wall:

CONCRETE (>C12/15)

FULL BLOCKS IN; GASBETON (>G4)

PLASTER, NATURAL LIGHT STONE

PERFORATED BRICKS IN; LATERITIOUS (>HLZ 12)

LIME (>KSL 12)

WARNING: excluded from use are perforated blocks in concrete (Hbl) and light concrete (Hbn).

SOLID BRICKS IN LATERITIOUS (>Mz 12)

LIME (>KSL 12), NATURAL COMPACT STONE, GYPSUM / DRYWALL OVER A WOOD STUD, PLYWOOD BACKER BOARD

CLEANING AND MAINTENANCE

DO NOT USE abrasive liquid detergents. Chlorine (Bleach) does not improve the aseptic aspect of the equipment and could lead to opacity and deterioration of the material.

DO NOT USE hot air with a temperature over 80°C (175°F) for drying the device.

In case of any doubts, refer to the technical department of Ponte Giulio

The device is designed and manufactured with the unique aim of easing the movement of aged or disabled people. For a proper use the device must be left free of any objects and easily accessible.

For further information, please visit our website or contact the Ponte Giulio technical department

Rev.0 – 15.04.2010