




G55JBR19
G55JBL19
G55JBL20
G55JBR20
G55JBL23
G55JBR23

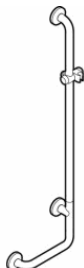


G55JOR01
G55JOL01

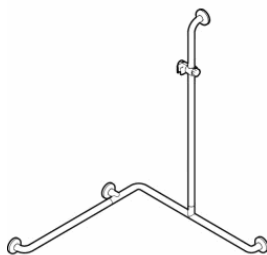


G55JBR04
G55JBL04

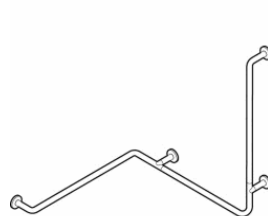

G55JBR06
G55JBL06

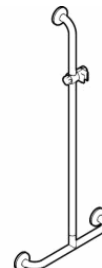

G55JBS16


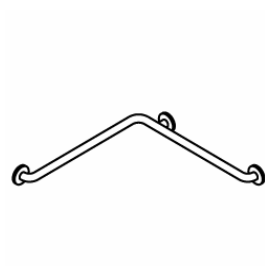








G55JBS01
G55JBS02
G55JBS03
G55JBS04
G55JBS05
G55JBS06
G55JBS07




G55JAS10
G55JAS11

ASSEMBLY INSTRUCTIONS AND MAINTENANCE

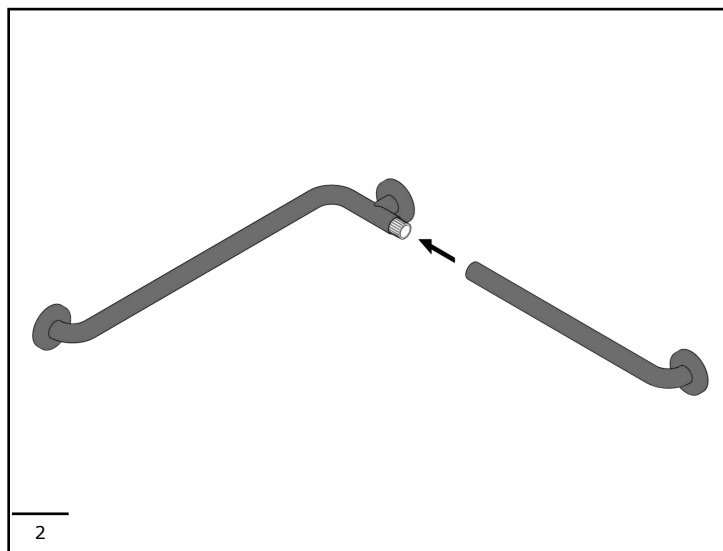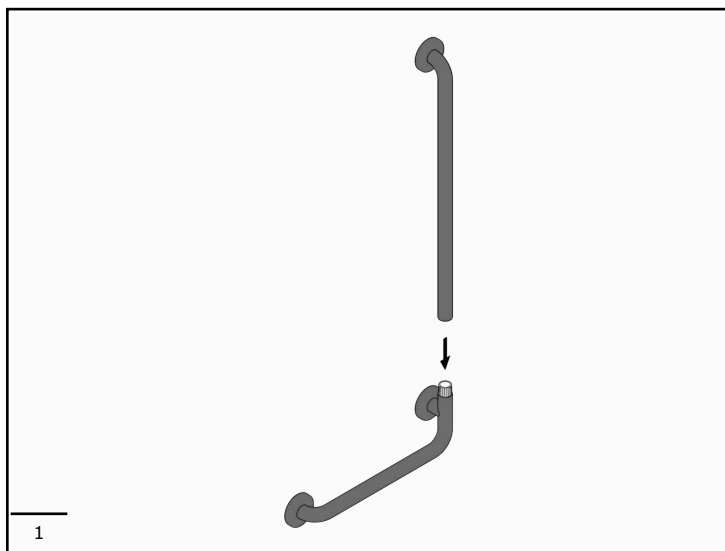

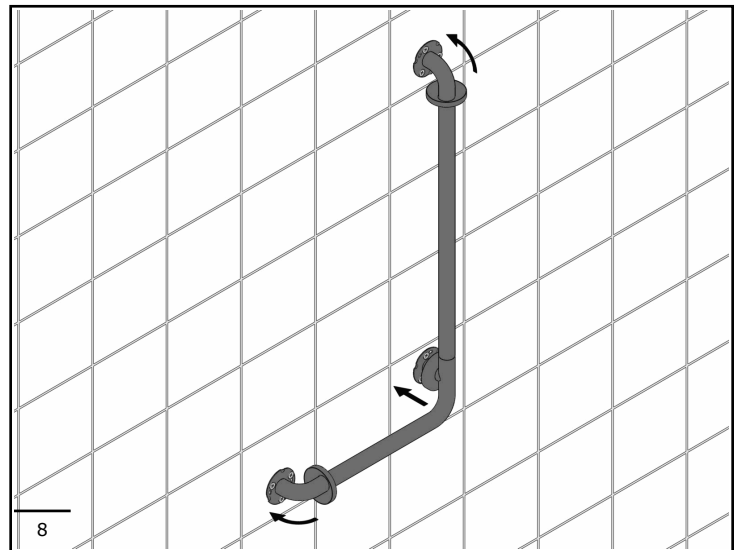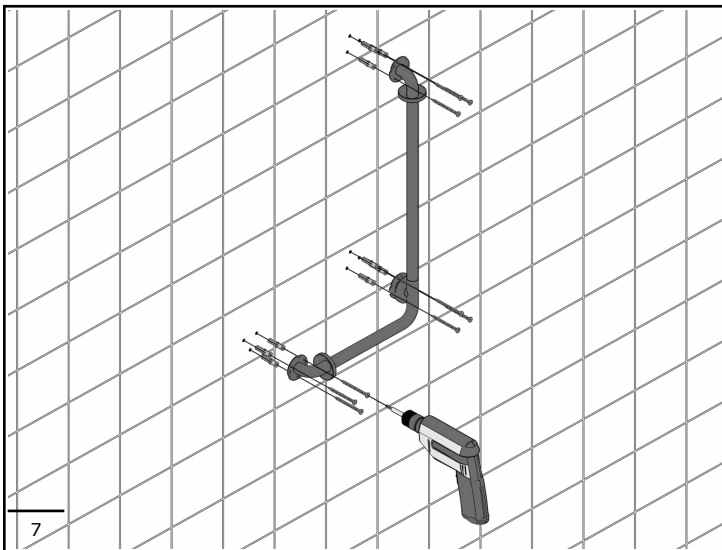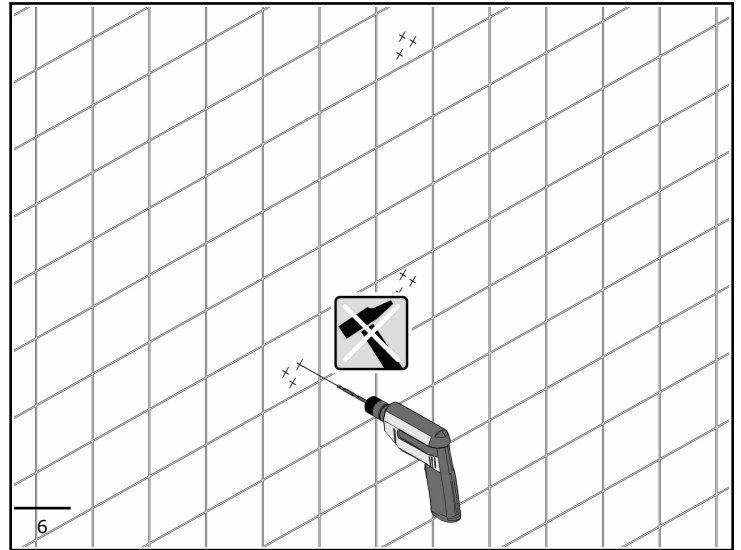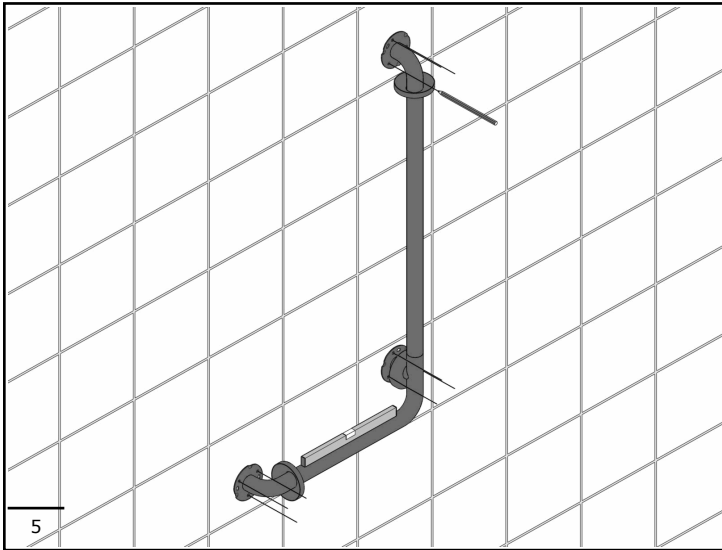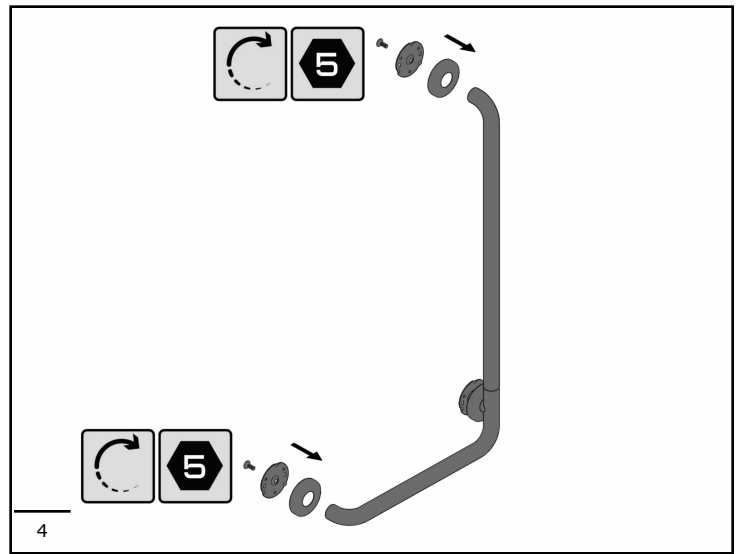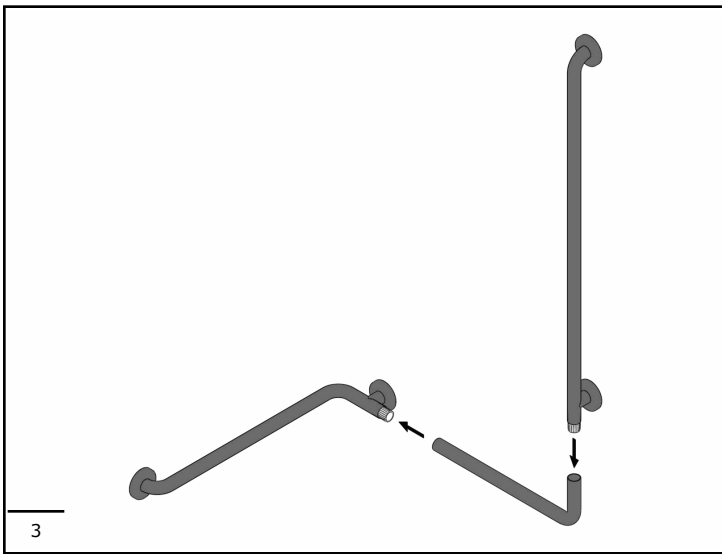

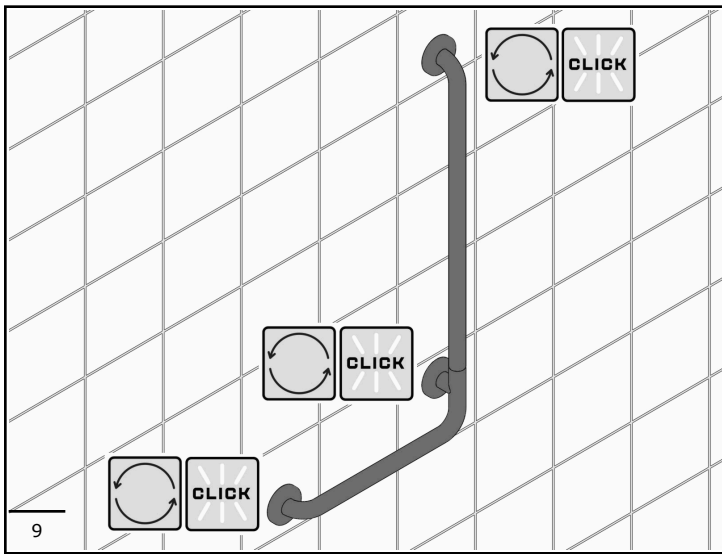

INSTRUCTIONS AND PRECAUTIONS FOR SAFE USE

- Prior to installing and using this grab bar, carefully read all the instructions.
- Retain this copy of the safety manual for future reference.
- Manufacturer, Ponte Giulio USA can be reached at 1-877-766-8301 or www.PonteGiulio.com
- This grab bar is designed to assist the user to sit comfortably, assist user to stand from a seated position, assist user in a slippery and wet environment, give user a secure place to hold, help user maintain their balance.
- Ensure this grab bar is not obstructed by other objects and is easily reachable.

GENERAL INFORMATION

- Keep packaging away from children to avoid risk of suffocation
- All parts of this grab bar are recyclable or reusable. It does not contain carcinogenic, mutagenic or toxic substances.
- This grab bar is capable of withstanding corrosion and temperature variations between 39 to 104 degrees F.

SAFETY AND INSTALLATION

- Do not exceed load capacity. Exceeding load capacity can cause damage to the product and harm to people.
- This grab bar should be installed by a competent installation professional.
- Ensure this grab bar is installed into a secure wall. Confirm that each fastener is securely mounted into wood studs or plywood backing/blocking material.
- Wall anchors or toggle bolts into drywall are not a secure mounting. This grab bar needs to be installed into studs or blocking.
- Do not install in a sauna.
- After installation, check to be sure unit has correct tension, moves up and down well but does not drop on it's own. There is a tension adjuster that can be tightened or loosened with a set key in the hinge mechanism.

MAINTENANCE

- Check grab bar for proper function prior to first use
- Inspect grab bar at least every 3 months to be sure it is secure and working properly.
- If grab bar becomes unstable, suspend use until it is properly checked by a competent professional.
- Tension hinge can be tightened or loosened, as needed, every time grab bar is inspected.

CLEANING

- Ensure this grab bar is free of grease and oils prior to use.
- Clean the surface using warm water, mild soap and a soft cloth. Do not use abrasive cleaners, chlorine bleach, strong acids or bases which may cause the material to dull.
- Do not use hot air, like a heat gun or blow dryer, for drying.

DISPOSAL

- Recycle unit when removed. All parts are recyclable or reusable.