



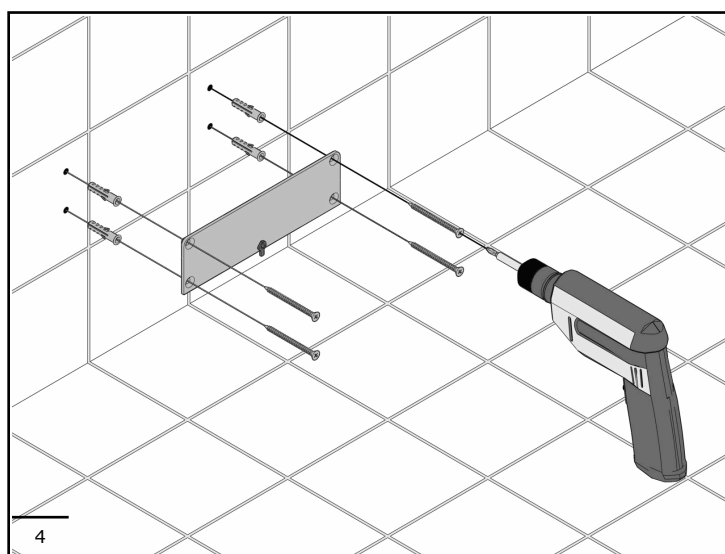
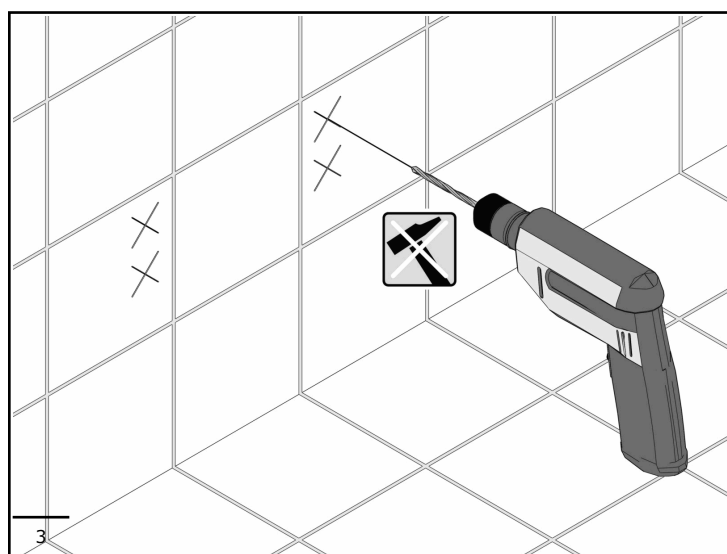
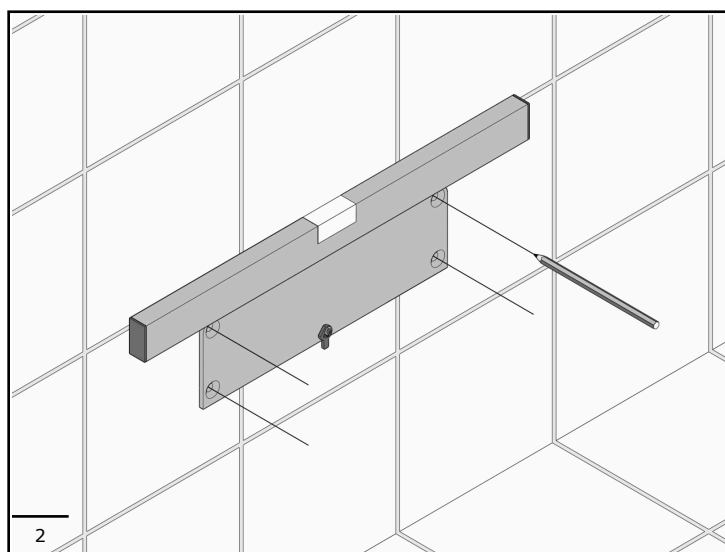
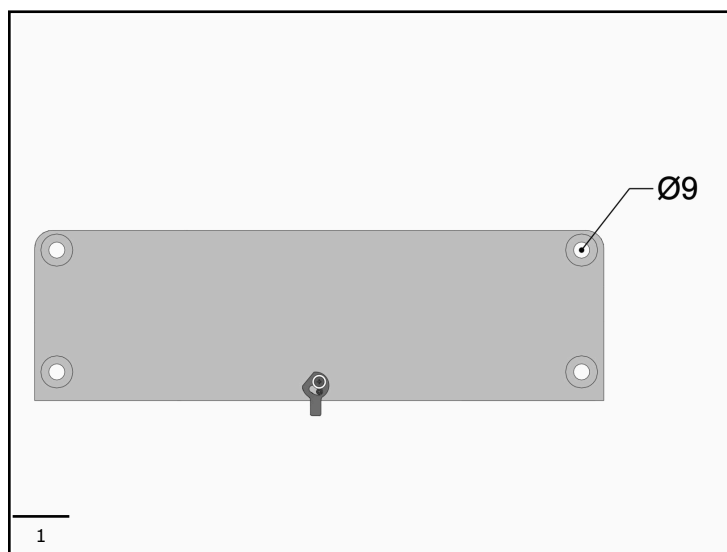
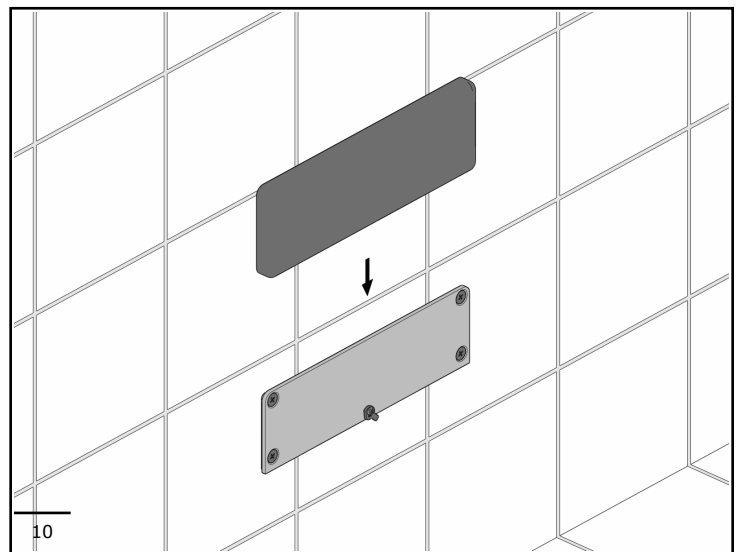
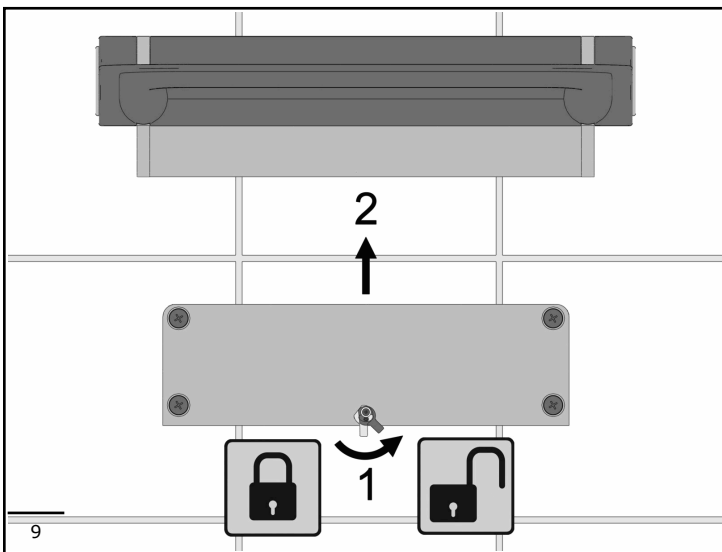
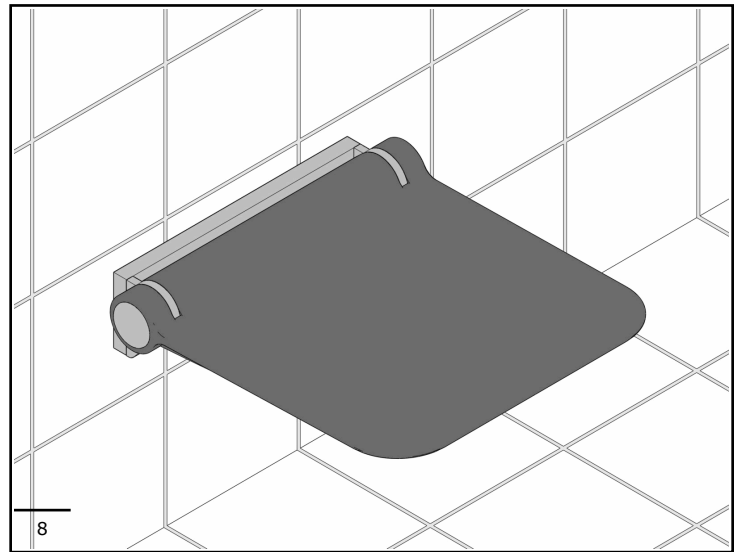
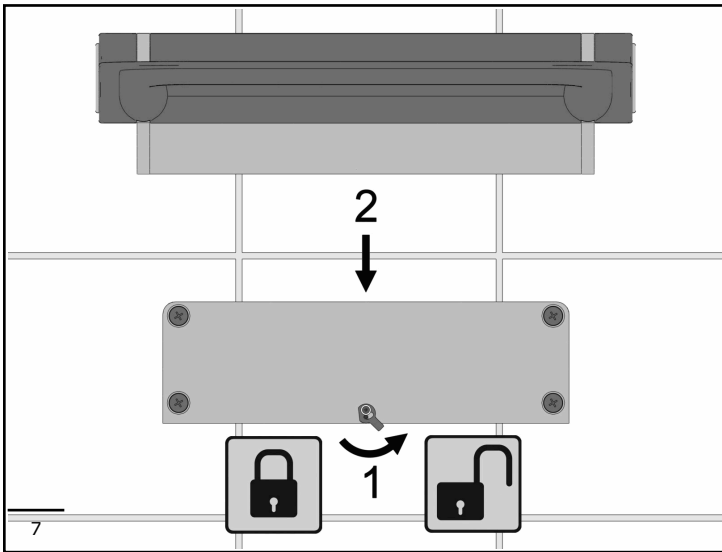
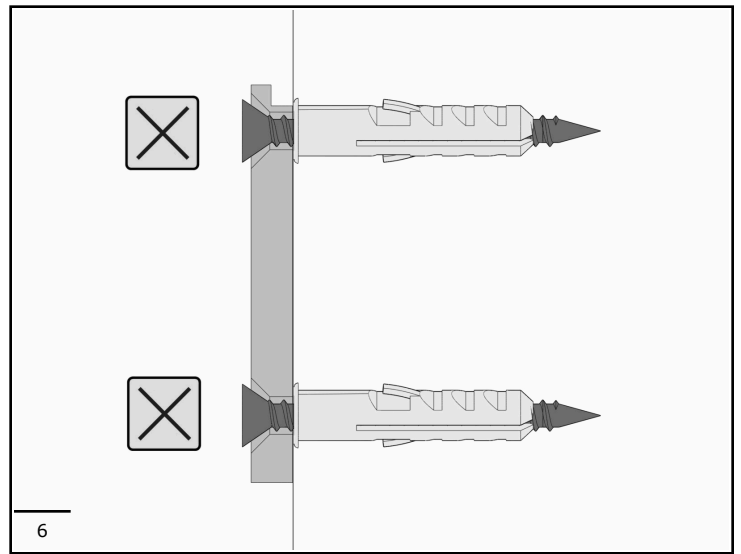
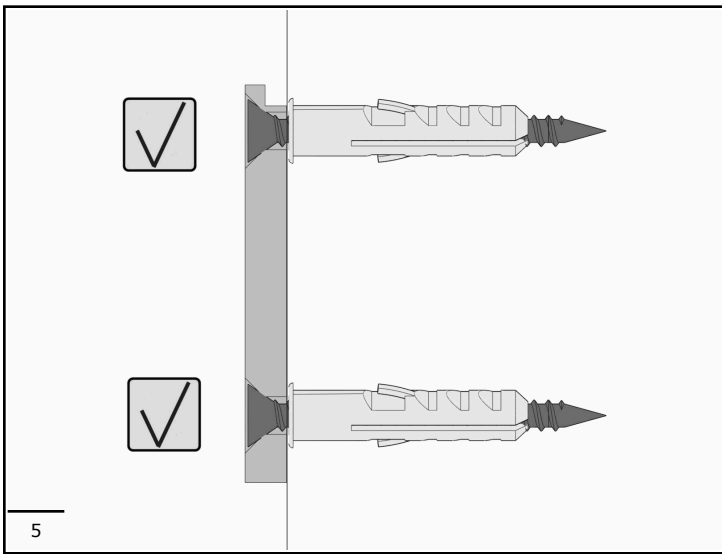
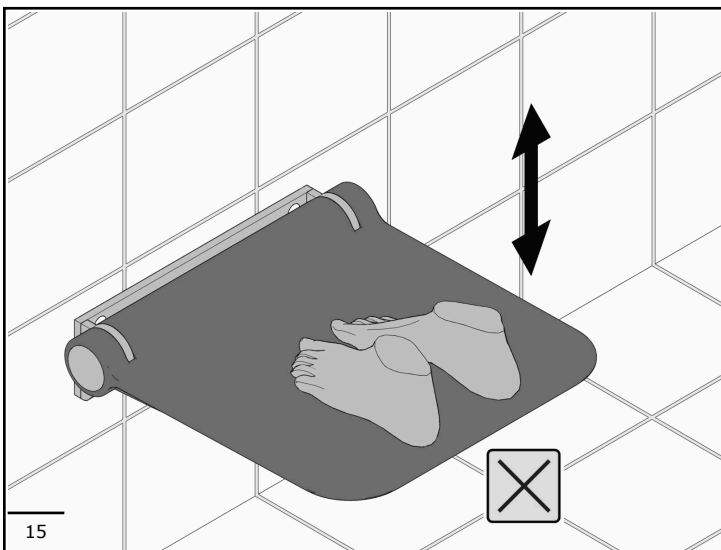
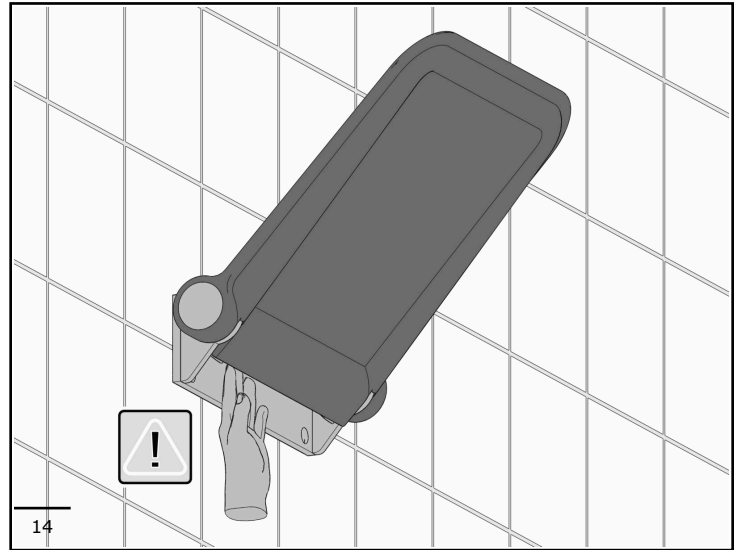
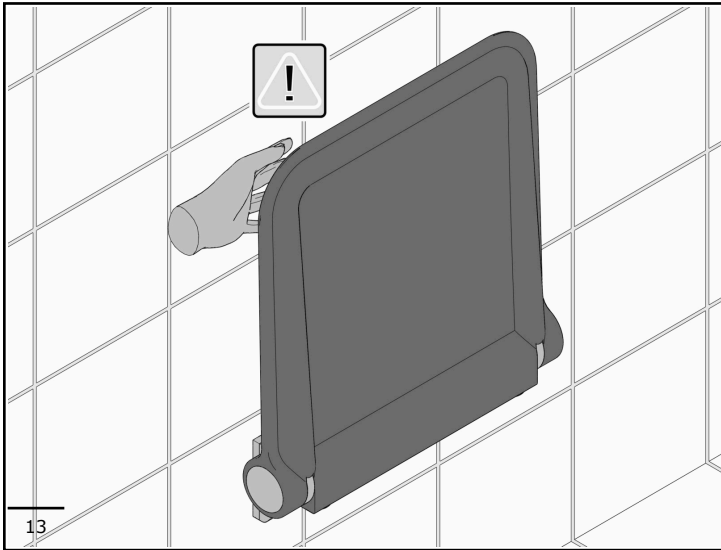
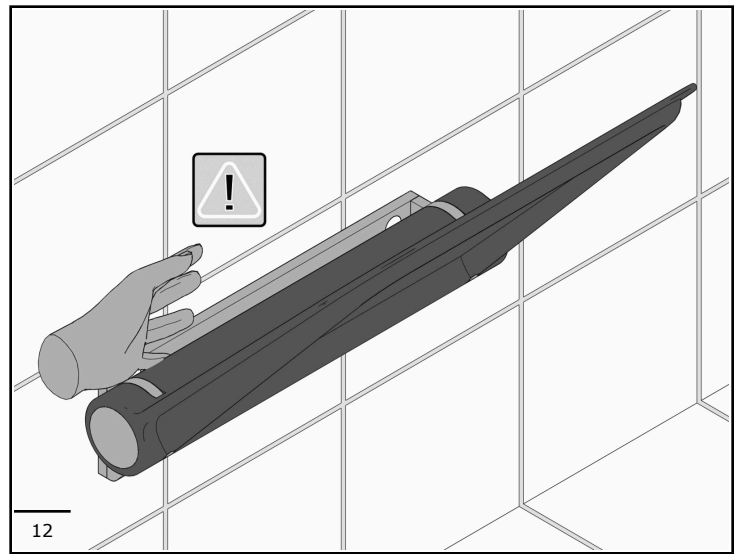
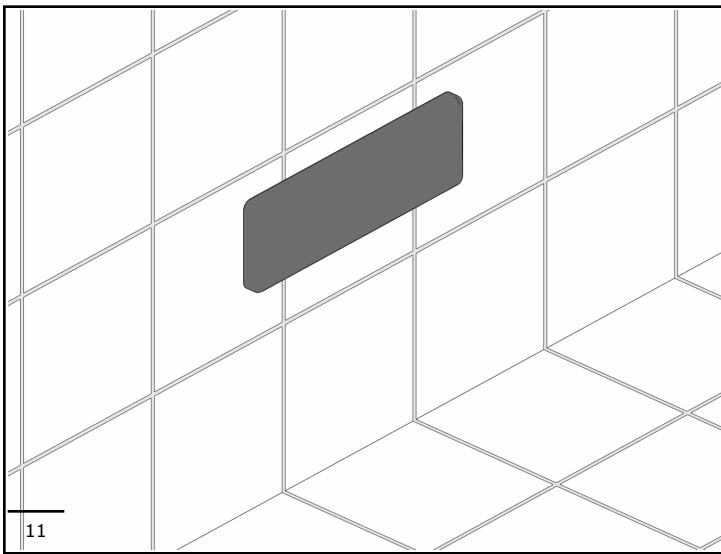
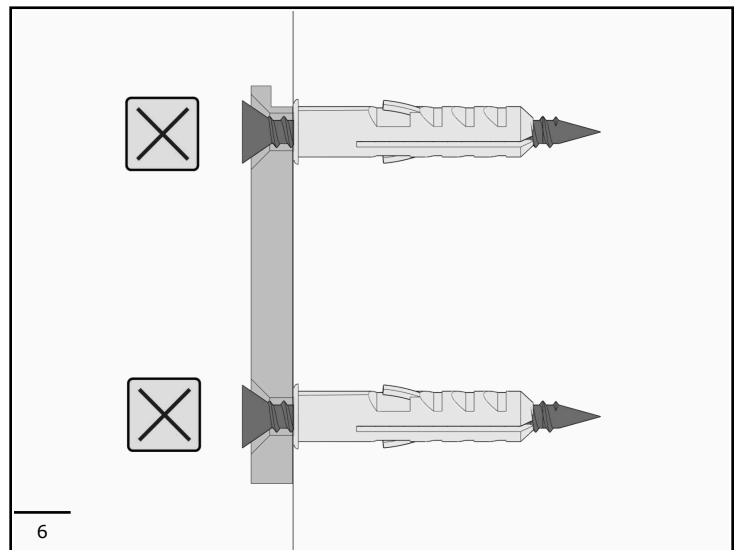
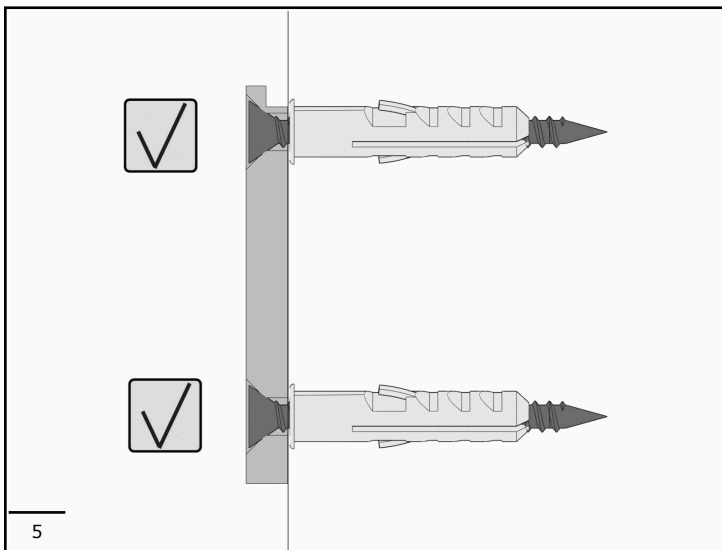








G12JDS08



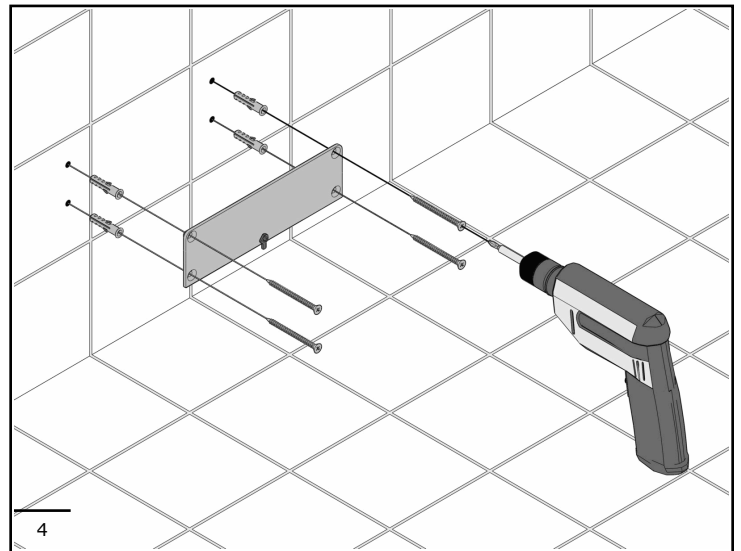


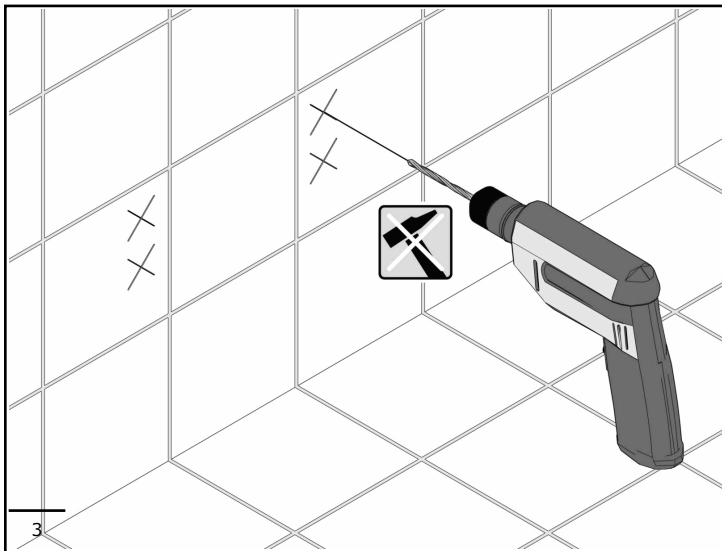

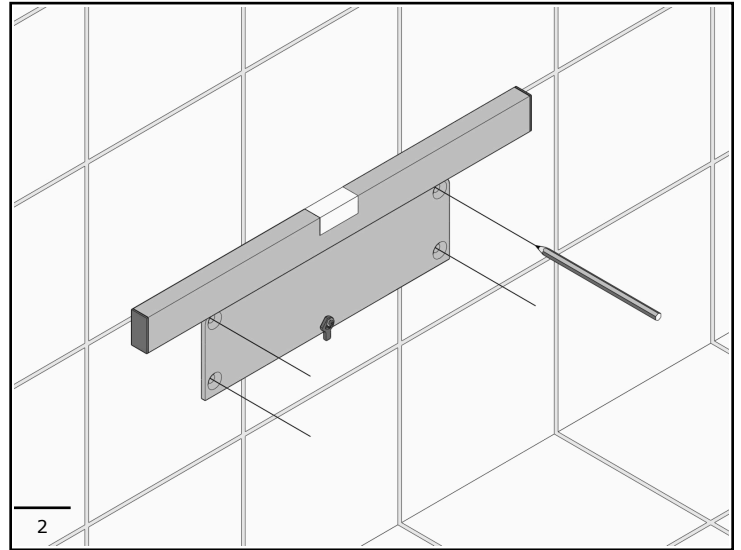

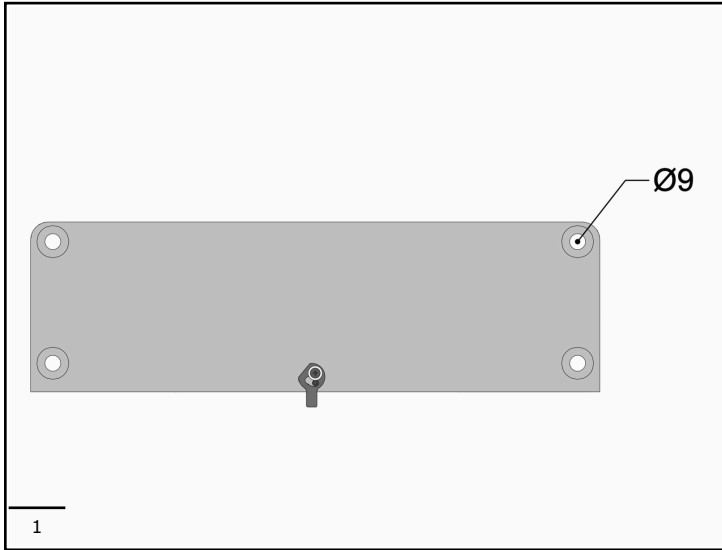

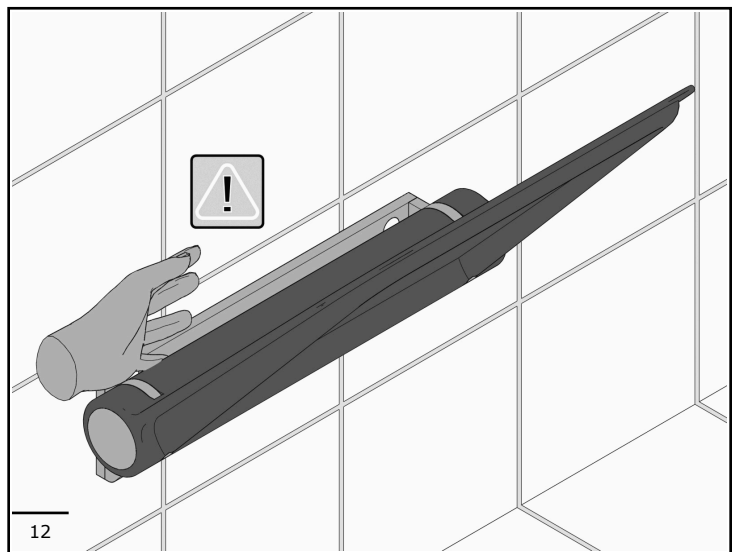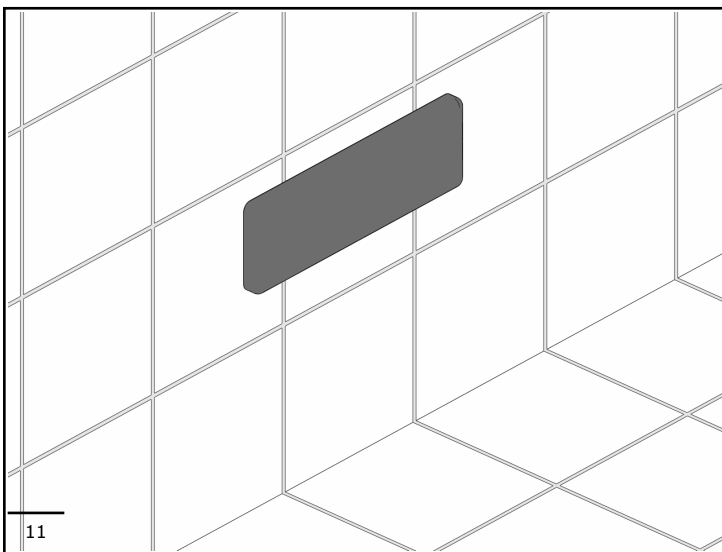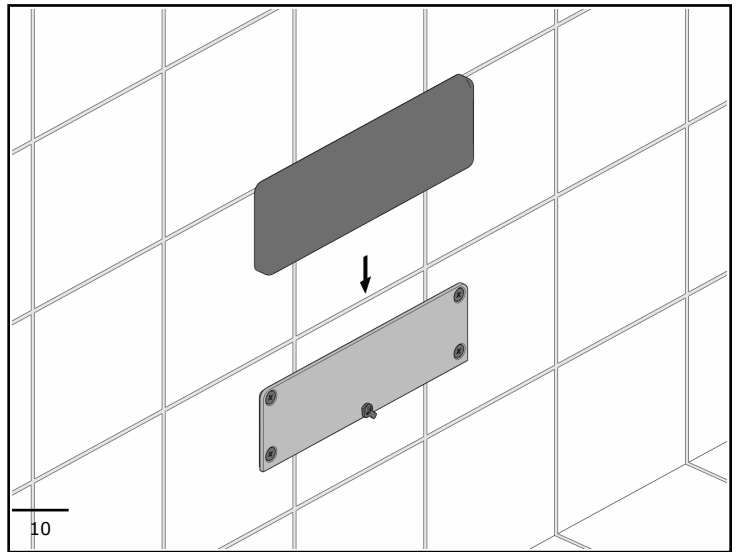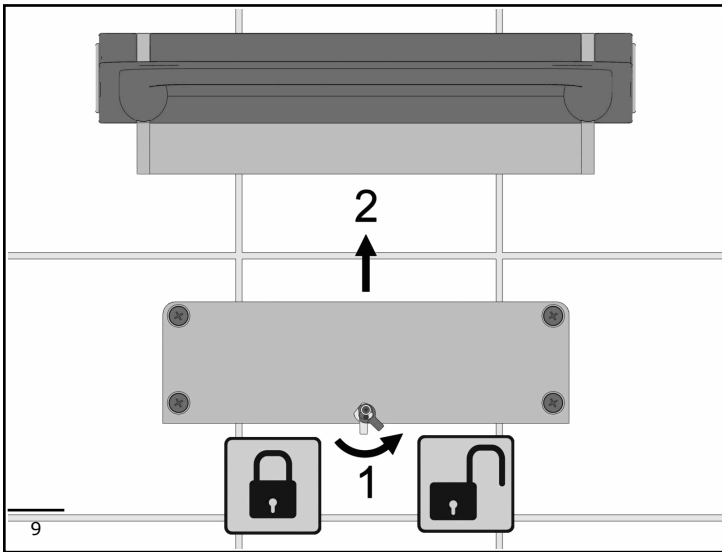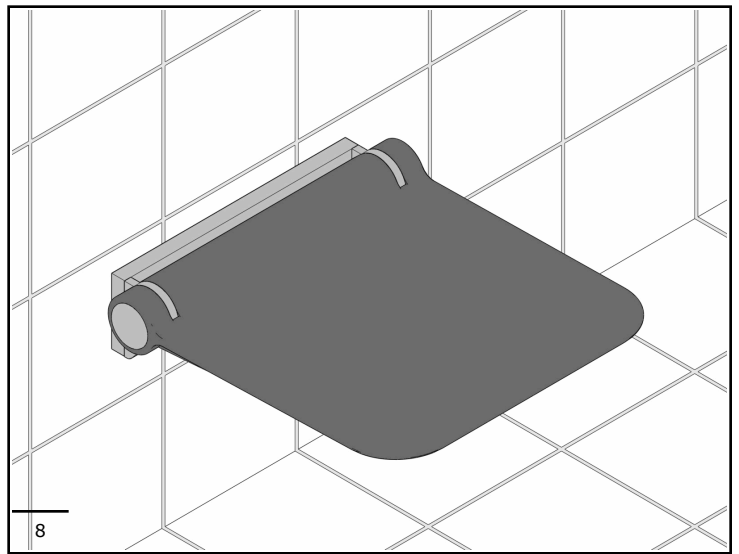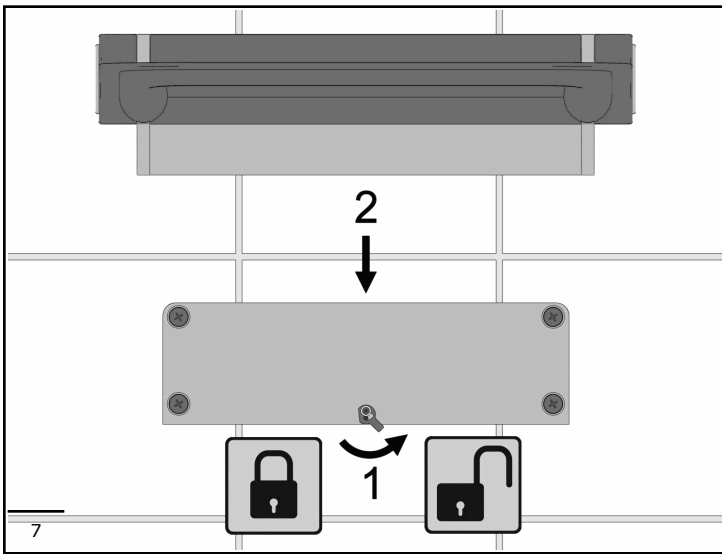

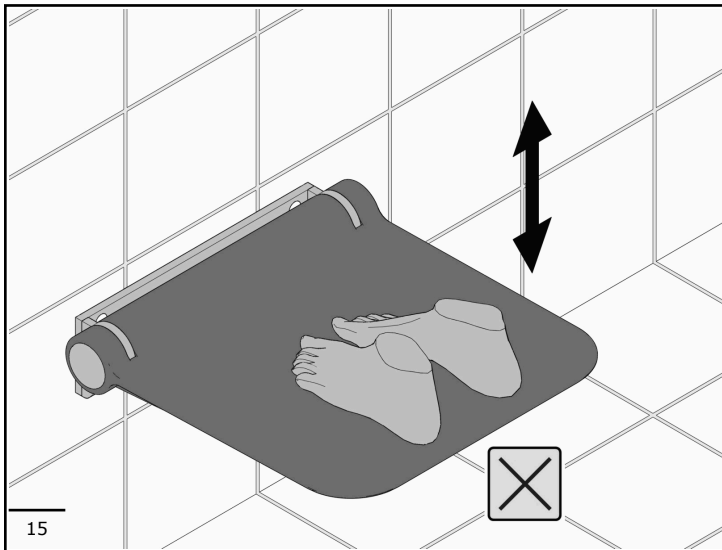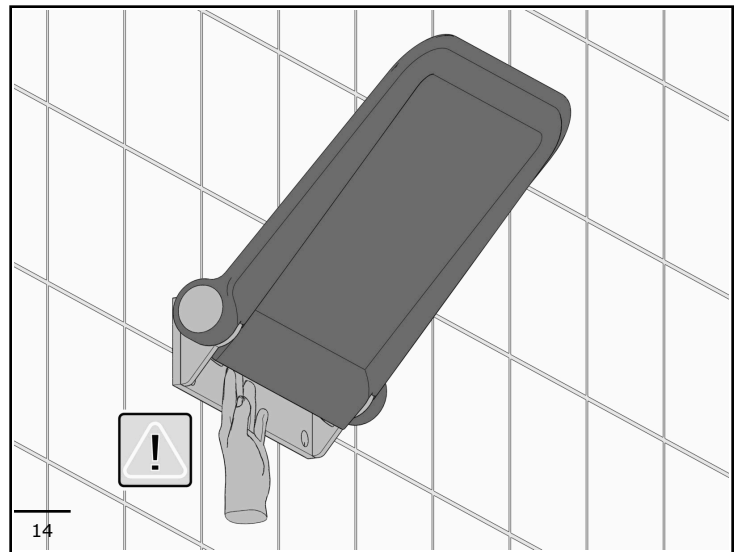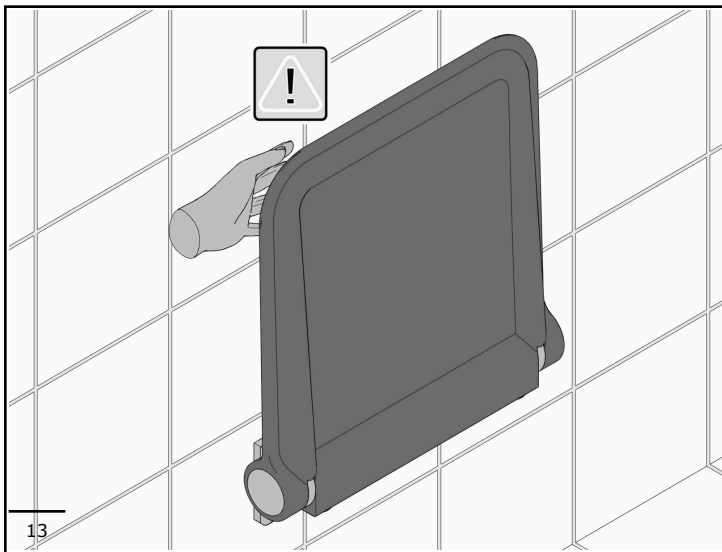

INSTRUCTIONS FOR USE

The device is a seat exclusively for sitting. Using the device improperly, for example for gymnastics exercises, can cause injuries with serious consequences. Do not subject the device to loads exceeding for which it was designed. Exceeding this load can cause permanent damage to the device or harm to people. To ensure correct use of the device, the latter must be left free of any objects and be easily reachable.

GENERAL INFORMATION

- Keep packaging away from children to avoid risk of suffocation
- All parts of this grab bar are recyclable or reusable. It does not contain carcinogenic, mutagenic or toxic substances.
- This grab bar is capable of withstanding corrosion and temperature variations between 39 to 104 degrees F.

SAFETY AND INSTALLATION

- Do not exceed load capacity. Exceeding load capacity can cause damage to the product and harm to people.
- This grab bar should be installed by a competent installation professional.
- Ensure this grab bar is installed into a secure wall. Confirm that each fastener is securely mounted into wood studs or plywood backing/blocking material.
- Wall anchors or toggle bolts into drywall are not a secure mounting. This grab bar needs to be installed into studs or blocking.
- Do not install in a sauna.
- After installation, check to be sure unit has correct tension, moves up and down well but does not drop on it's own. There is a tension adjuster that can be tightened or loosened with a set key in the hinge mechanism.

ROUTINE AND MAINTENANCE

- Check the stability of the wall-mount at least once every 3 months.
- Check the grab bar flange tension, at least every 3 months. If it pivots freely, tighten the tension pin until the movement of the arm occurs only when user is adjusting it. A quarter turn of the set key is usually enough to either tighten or loosen the tension.
- Should the grab bar become unstable, contact the installer.
- This grab bar has a lifetime warranty against defect in materials or workmanship.

CLEANING

- Ensure this grab bar is free of grease and oils prior to use.
- Clean the surface using warm water, mild soap and a soft cloth. Do not use abrasive cleaners, chlorine bleach, strong acids or bases which may cause the material to dull.
- Do not use hot air, like a heat gun or blow dryer, for drying.

DISPOSAL

- Recycle unit when removed. All parts are recyclable or reusable.