





H51GHS41



**ASSEMBLY INSTRUCTIONS AND MAINTENANCE**

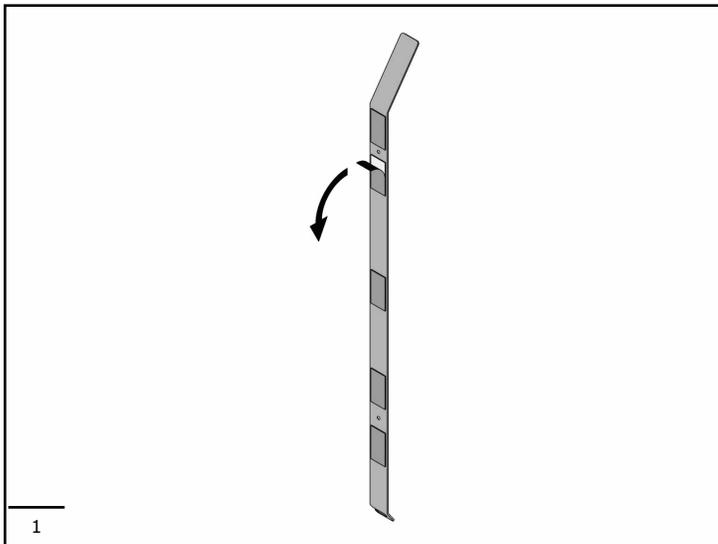





INSTALLATION WITH DOUBLE-SIDED ADESIVE

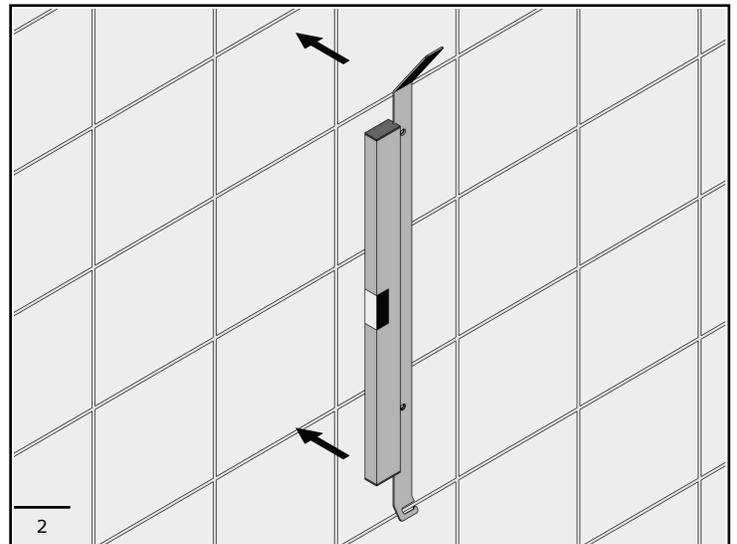





With the supplied cleaning towel, wipe the tiles in correspondence of the adhesive and wait a few seconds for a perfect evaporation of the liquid. Remove the protective film of the double-sided adhesive taking care not to touch the double-sided adhesive with your hands.  
**WARNING!** The double-sided adhesive to be applied only on smooth surfaces like tile or any other, not on concrete.

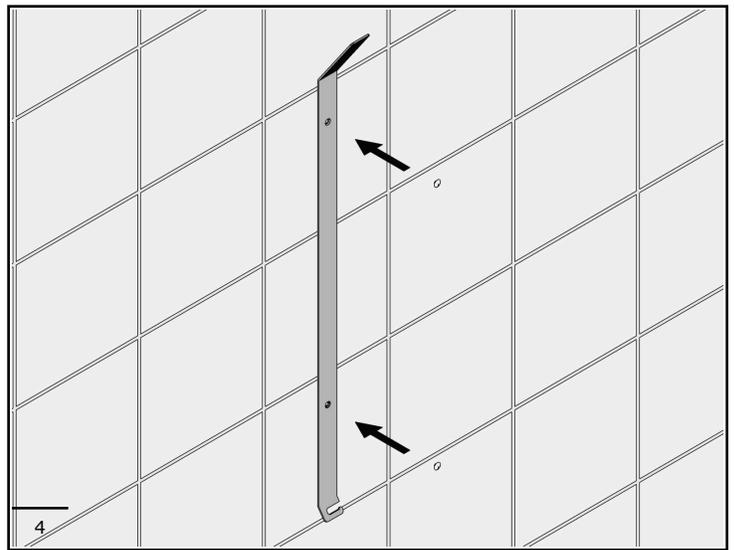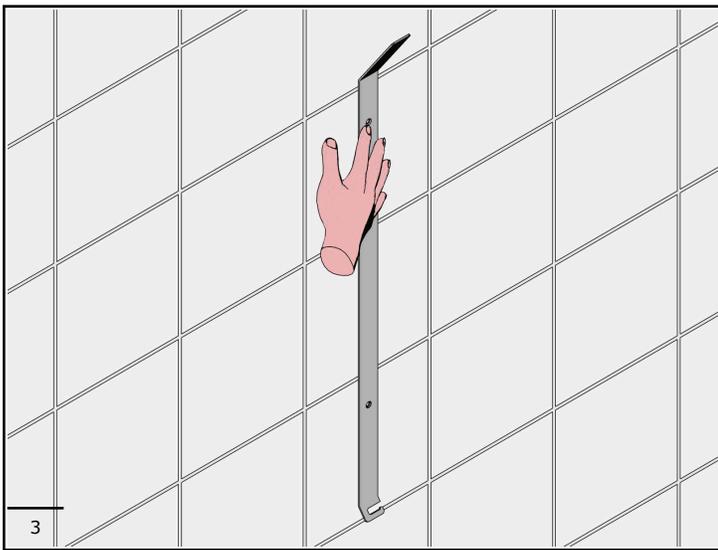

Press on the bar at the height of the double-sided adhesive to make it adhere perfectly to the wall. For a perfect fit we recommend waiting at least 1 hour before using the shower.

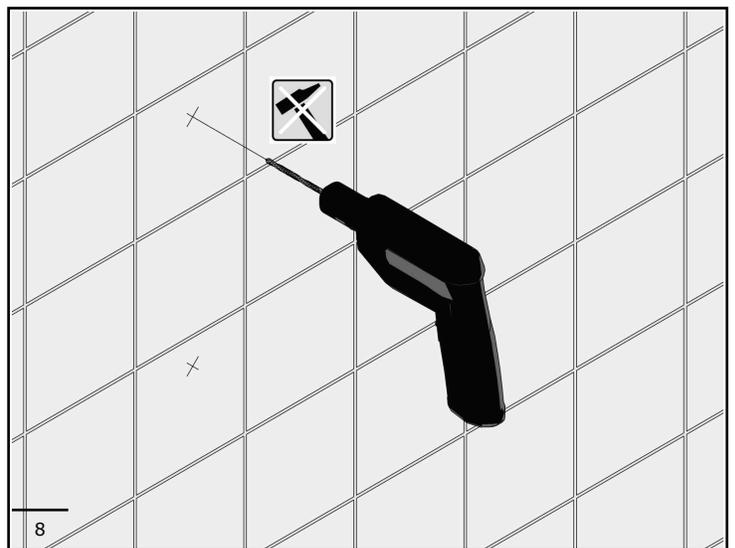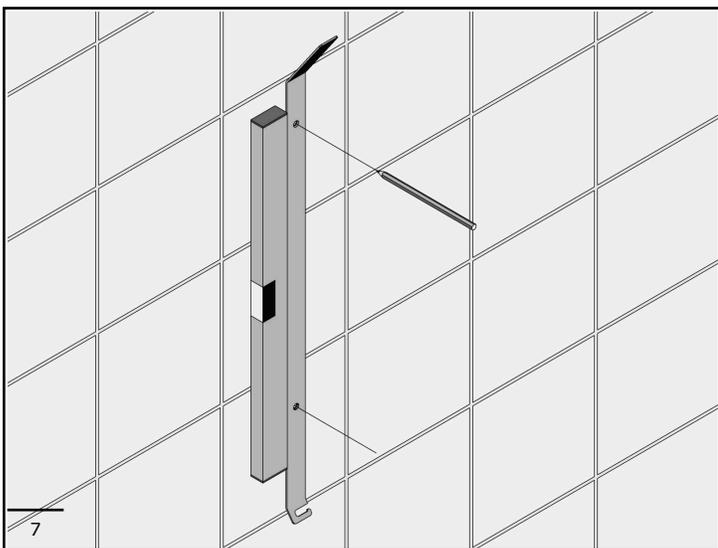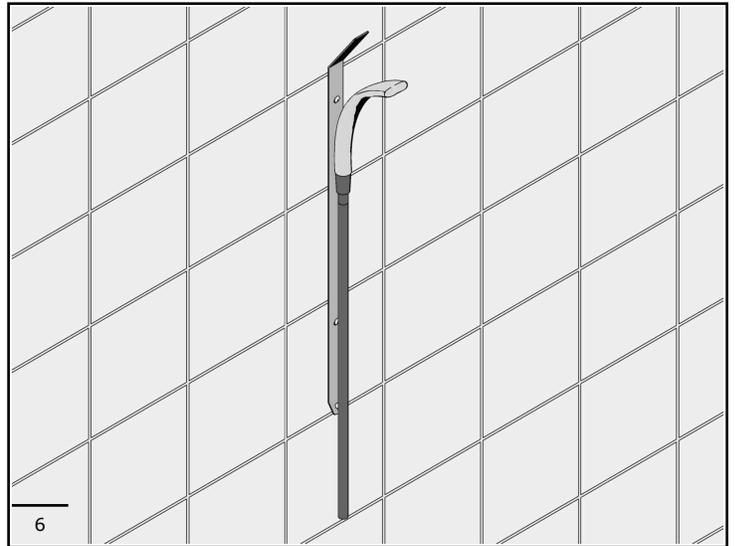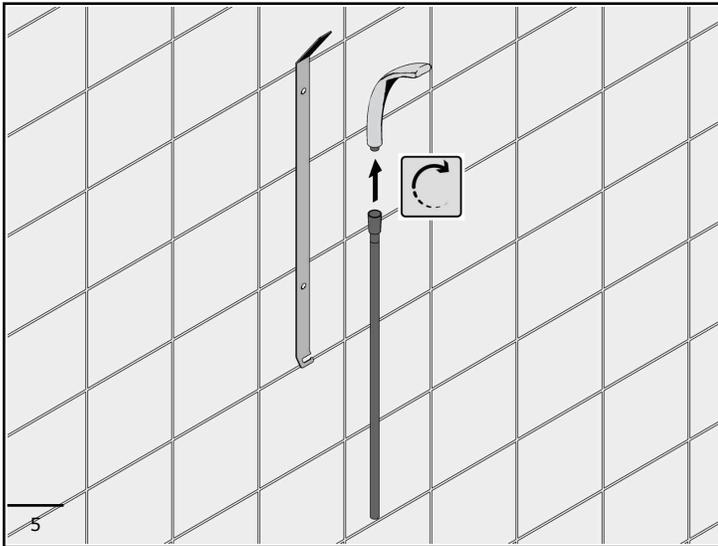


INSTALLATION WITH SCREWS AND DOWELS

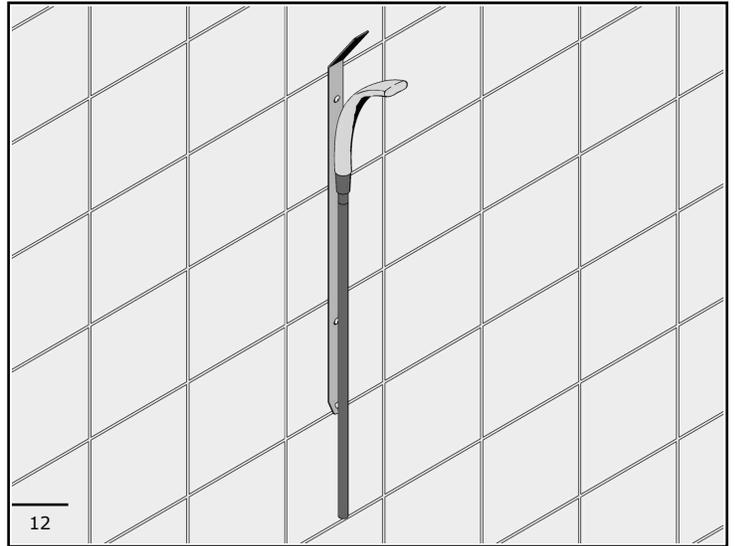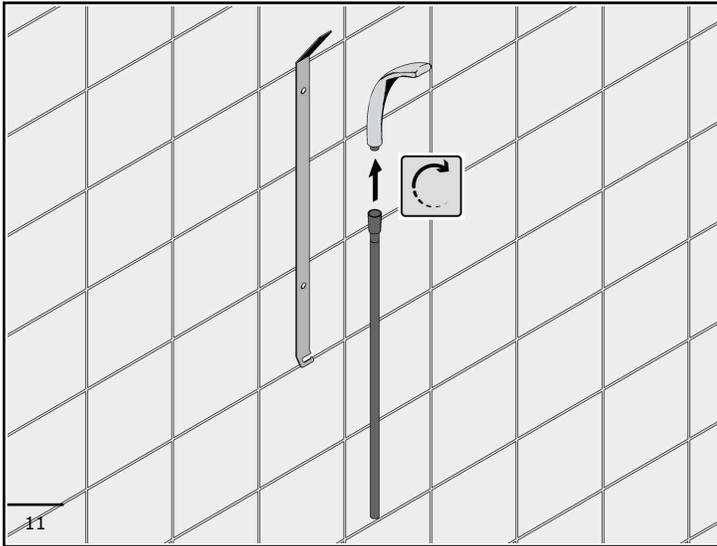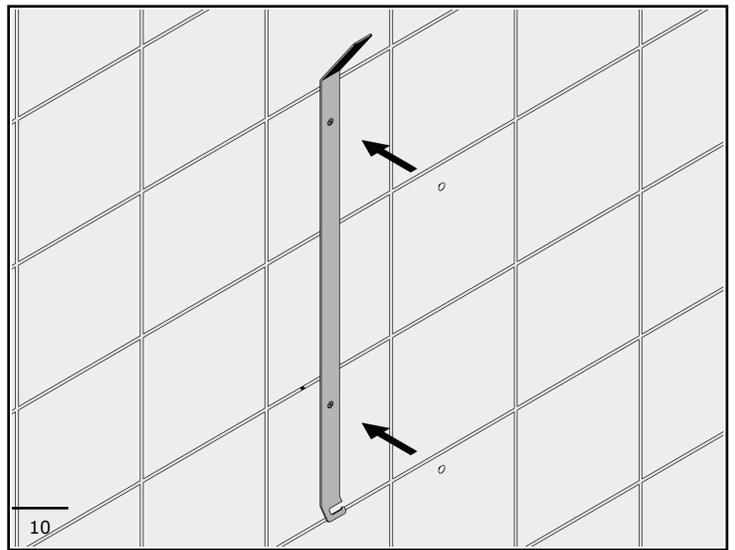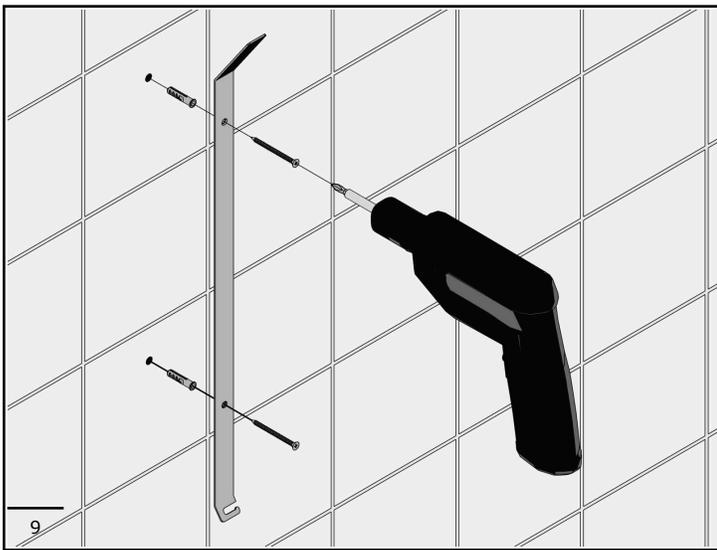

**WARNINGS FOR RESIDUAL RISKS**



This product contains a permanent magnet.

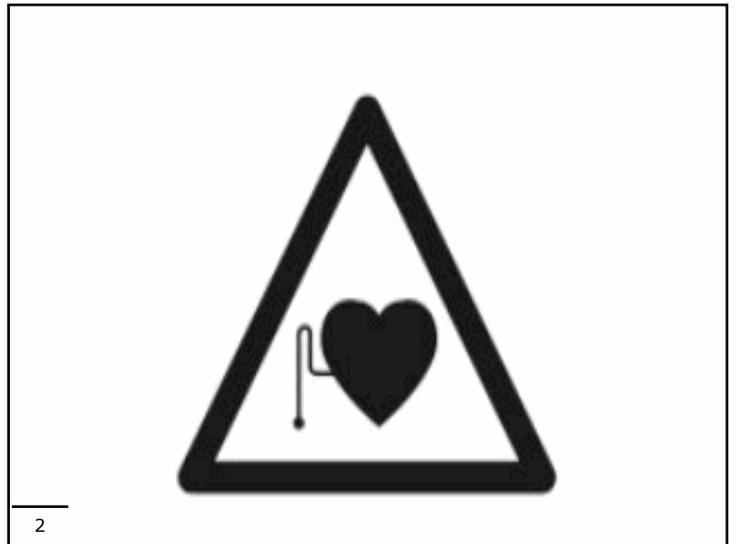

The magnets may affect the operation of some implanted cardiac pacemakers and defibrillators, if you wear one of these devices keep a safe distance of at least 4 in.

**GENERAL INFORMATION**

- Keep packaging away from children to avoid risk of suffocation

- All parts of this grab bar are recyclable or reusable. It does not contain carcinogenic, mutagenic or toxic substances.
- This grab bar is capable of withstanding corrosion and temperature variations between 39 to 104 degrees F.

#### **SAFETY AND INSTALLATION**

- Do not exceed load capacity. Exceeding load capacity can cause damage to the product and harm to people.
- This grab bar should be installed by a competent installation professional.
- Ensure this grab bar is installed into a secure wall. Confirm that each fastener is securely mounted into wood studs or plywood backing/blocking material.
- Wall anchors or toggle bolts into drywall are not a secure mounting. This grab bar needs to be installed into studs or blocking.
- Do not install in a sauna.
- After installation, check to be sure unit has correct tension, moves up and down well but does not drop on it's own. There is a tension adjuster that can be tightened or loosened with a set key in the hinge mechanism.

#### **SAFETY PRECAUTIONS**

The installation of the device does not present particular difficulties, check that the fixing elements, if supplied, are suitable for the installation wall.

#### **MAINTENANCE**

- Check grab bar for proper function prior to first use
- Inspect grab bar at least every 3 months to be sure it is secure and working properly.
- If grab bar becomes unstable, suspend use until it is properly checked by a competent professional.
- Tension hinge can be tightened or loosened, as needed, every time grab bar is inspected.

#### **CLEANING**

- Ensure this grab bar is free of grease and oils prior to use.
- Clean the surface using warm water, mild soap and a soft cloth. Do not use abrasive cleaners, chlorine bleach, strong acids or bases which may cause the material to dull.
- Do not use hot air, like a heat gun or blow dryer, for drying.

#### **DISPOSAL**

- Recycle unit when removed. All parts are recyclable or reusable.