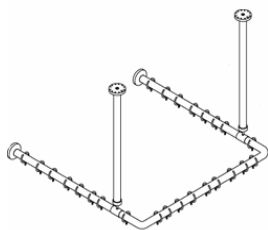
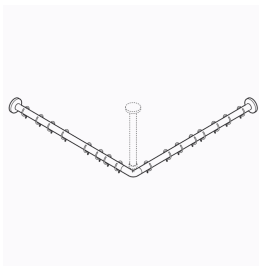
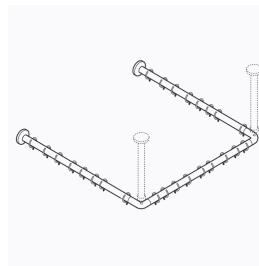
G40JRS11

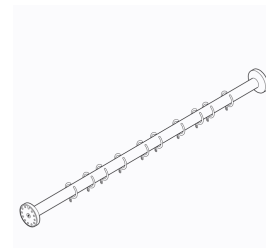
G40JRS22

G01JRS10
G01JRS11
G01JRS12
G01JRS13
G01JRS14
G01JRS15

G01JRS20
G01JRS21
G01JRS22
G01JRS23
G01JRS24

G01JRS01
G01JRS02
G01JRS03
G01JRS04
G01JRS05
G01JRS06
G01JRS07
G01JRS08
G01JRS09

IT

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EN

INSTRUCTIONS AND PRECAUTIONS FOR SAFE USE.

Prior to installing and using the device, carefully read the instructions for use and – in particular – the safety precautions, and comply with them. Store this manual for reference for the entire life-cycle of the device. Should this manual get damaged, or if certain sections are difficult to understand or give rise to doubts, prior to using the device contact the company at the address indicated on the last page. Any changes to the information provided in the instruction manual – if deemed important for the user's safety – shall be announced through the Ponte Giulio Internet website and notified to all authorised dealers.

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EN

GENERAL INFORMATION

The device is safe to use, even for people with reduced visual, reading or cognitive abilities. There are no circumstances

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hampering ease of use of the product.
The device must not be dismantled and does not have dismantable parts.
The device's overall weight does not exceed 10 kg.
The device does not contain carcinogenic, mutagenic or toxic substances.
It is capable of withstanding corrosion and temperature variations.

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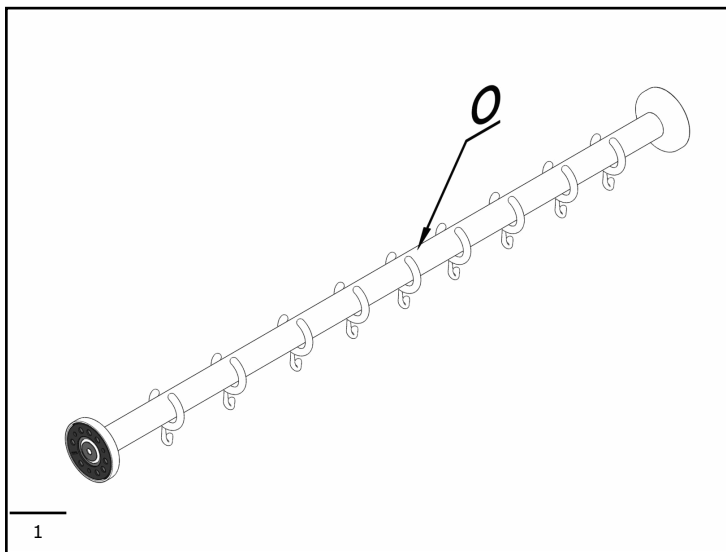
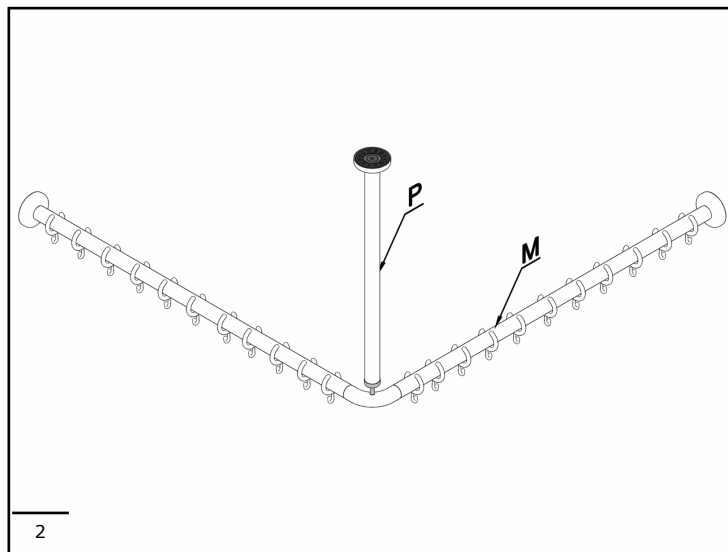
GB - The procedure described above is valid for cutting to size and correctly mounting the horizontal shower pipe(M)-(N)-(O), and the ceiling support (P).

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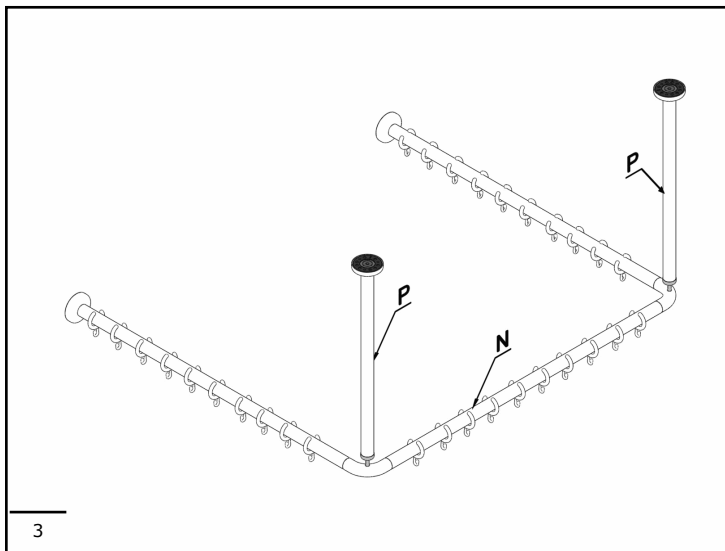
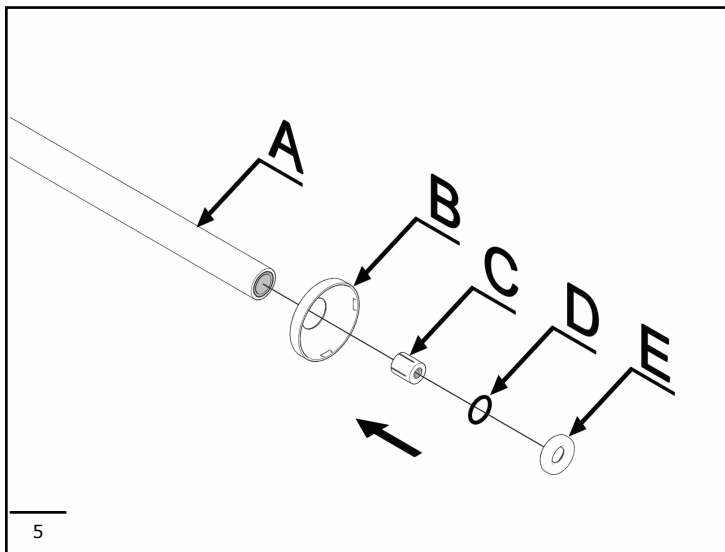
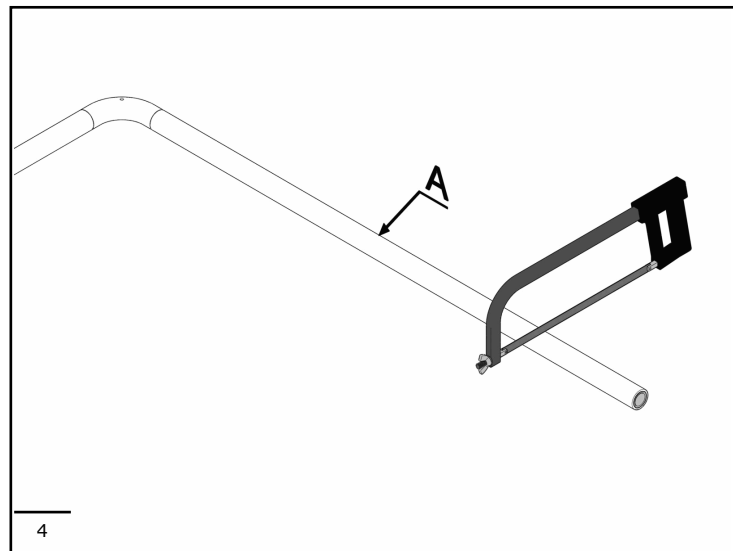
GB - Insert the flange-cover (B) in the shower curtain pipe [fig.5]. Insert the expansion insert (C) with the hole forth reading towards the exterior, then insert the washer (D) [fig.5] and attach the cap (E) to the end of the shower pipe to tighten the pipe (A) to the joint curve (I) [fig.7].

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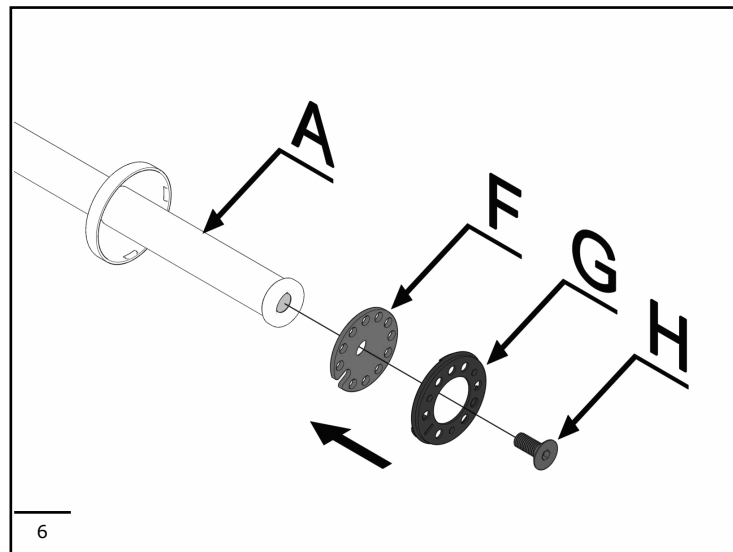
GB - Cut the shower curtain pipe (A) to size according to needs [fig.4].

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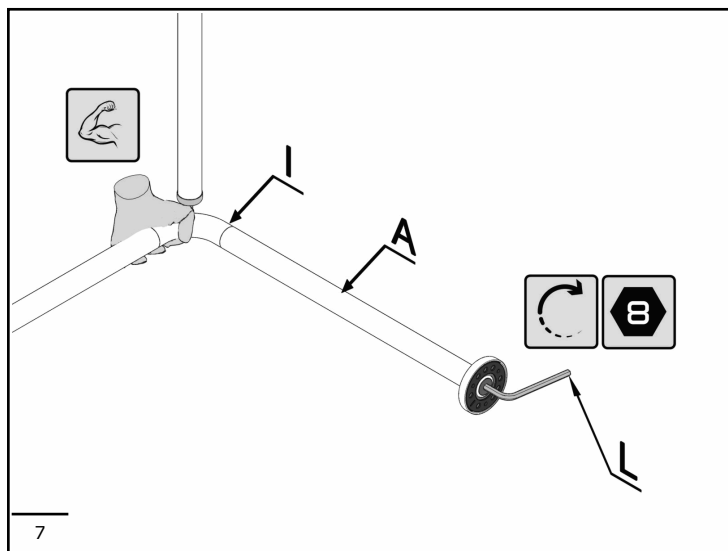
GB - Fix the flange (F) and the flange holder (G) to the pipe with the screw (H) [fig.6].

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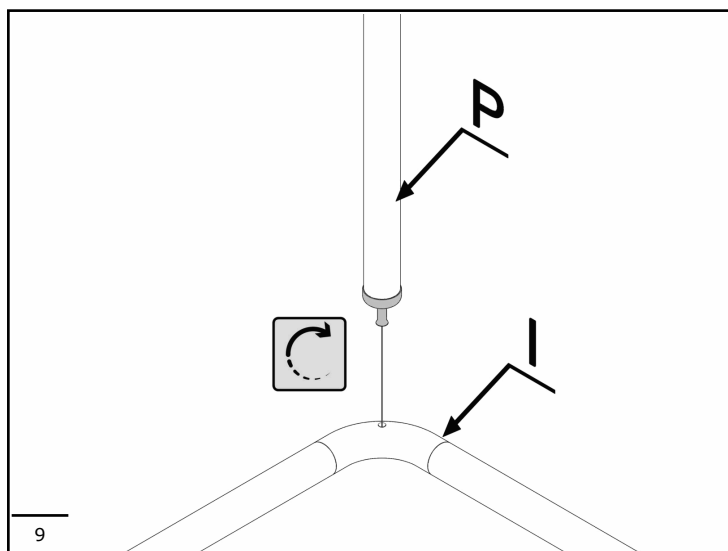
GB - Tighten the fixing kit with an 8 mm allen wrench (L) holding the curve tightly (I) so that the pipe doesn't turn [fig.7]. N.B. Once the fixing kit is tightened, if there is any air between the pipe (A) and the curve (I), turn the pipe by hand until it is perfectly even with the curve.

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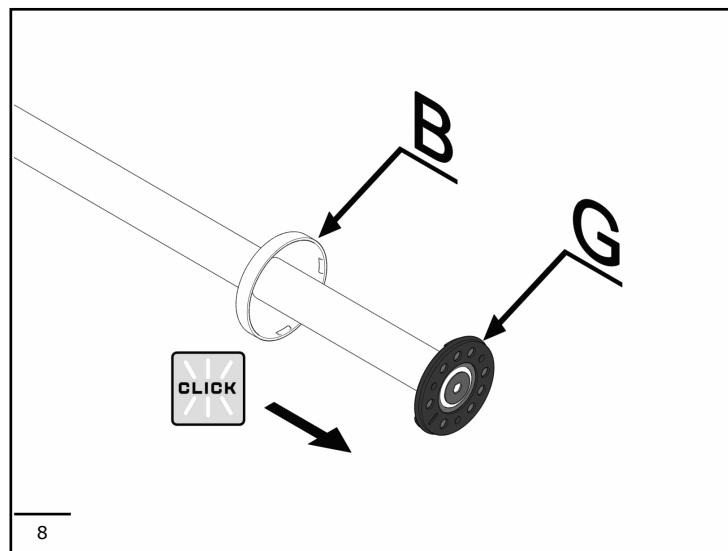
GB - Fix the support (P) in the slot in the curve (I) [fig.9].

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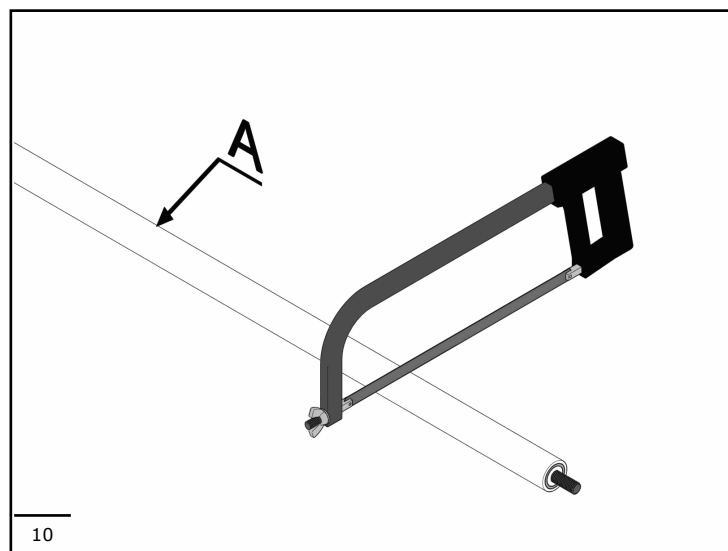
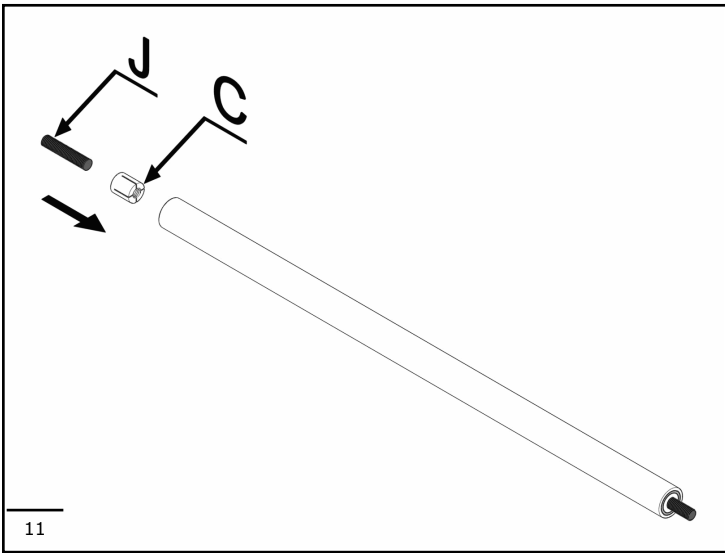
GB - Cut the desired size of both nylon and aluminium (A).

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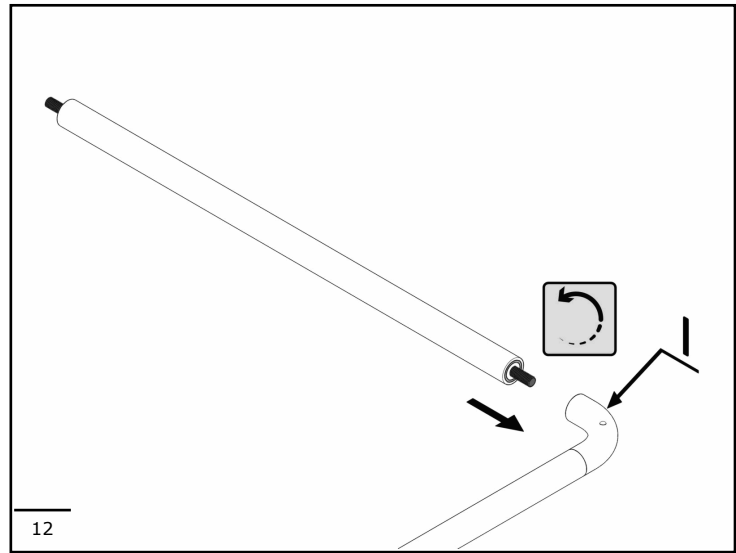
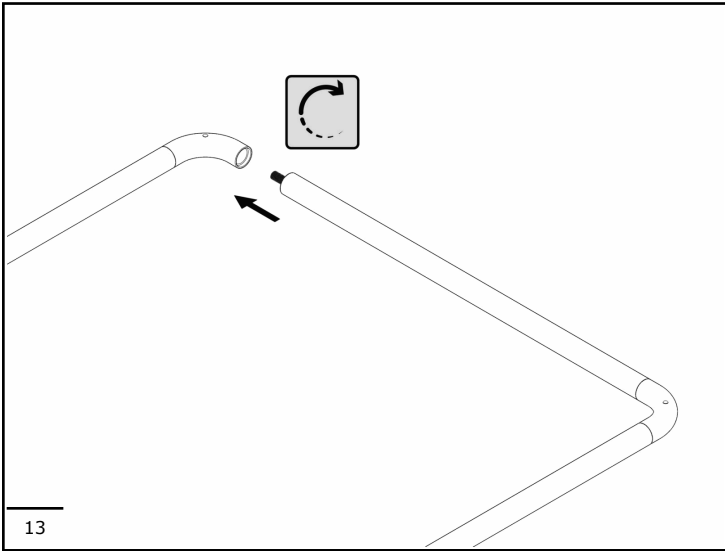
GB - Curved part already installed on curtain rail (l).

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GB - Shower curtain rings are made of very slippery material. However, to allow better flow of the shower curtain after the installation is recommended to clean the pipe.

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used exclusively as a shower curtain rod. Using the device improperly, for example for gymnastics exercises, can cause injuries with serious consequences. To ensure correct use of the device, the latter must be left free of any objects and be easily reachable.

EN

SAFETY PRECAUTIONS

The device must be installed by trained and qualified personnel (bricklayer or professional plumber) who must verify whether the fixing elements, if provided, are suitable for the relevant wall. To guarantee stability and safety, the device must be attached with wall plugs and screws suitable for the relevant type of wall. Do not subject the device to loads exceeding those specified. If the device should become unstable, suspend its use until it is properly checked by the relevant technician. The manufacturer declines all responsibility for installation of the device not conforming in this manual for use and maintenance.

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EN

CLEANING

Clean the surface of the device using a damp cloth wetted with liquid neutral soap (e.g. dish soap). It is forbidden to use abrasive liquid detergents containing chlorine, strong acids or bases, concentrated solvents such as acetone and trichloroethylene or derivatives, which may cause the material too dull. It is forbidden to use hot air sources for drying. Disinfect the device's surface using a bleach-type product at least once a week.

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EN**MAINTENANCE**

Check the stability of the fixing at least every 3 months.
Should the device become unstable, contact the installer.
The device has no adjustments.
It shall be fully replaced in case of any damages

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EN**CONDITIONS OF USE**

The device is not affected by the room's temperature, though it must NOT be installed in environments subject to temperatures below 4°C or above 40°C.
Do not install in saunas. Not comply with the indicated temperatures may damage the plastic coating of the device.

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EN**DISPOSAL**

The user is responsible for the device's disposal, which must occur in accordance with the laws of the country in which the device is installed. The device is not manufactured using substances that are hazardous to human health and the environment, and all parts comprising it are recyclable and/or reusable. In order to safeguard the environment, the device also contains a certain amount of recycled materials.
Keep away from children packaging; risk of suffocation.

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EN**FURTHER INFORMATION**

For further information, please visit our website or contact the our dealer zone

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